





HALYARD*

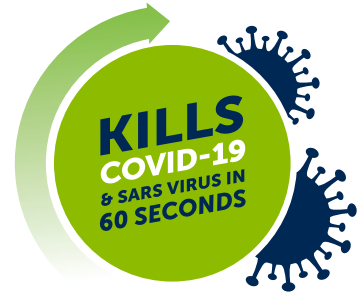

ISOWIPE*

HOSPITAL GRADE DISINFECTANT BACTERICIDAL WIPE

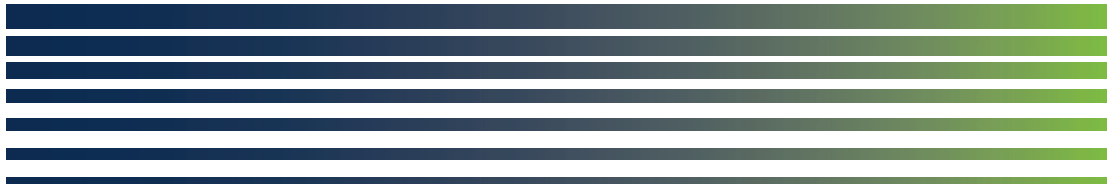

Kills Bacteria (*S. aureus*, *E. coli*, *Ps. aeruginosa*,
Pr. vulgaris, *Salmonella choleraesuis*)
Kills Fungi (Yeast & Mould)

REFILL PACK
42 cm x 14 cm

AUST L 342811	70% v/v Isopropyl alcohol impregnated wipes	Single Use Only		Store below 25°C	75 Wipes	REF 6836
---------------	---	-----------------	--	------------------	----------	----------

DISPENSING DIRECTIONS:

- Cut end of pack.
- Do not remove ISOWIPE* Bactericidal Wipes from wrapper.
- Stand refill pack in ISOWIPE* Bactericidal Wipes Container 6835.
- Locate centre sheet from middle of the roll and pull up.
- Using the end of blue cap, insert the corner of the first sheet in the dispensing hole.



DO NOT PUSH YOUR FINGER THROUGH THE DISPENSING HOLE.

- Replace the large blue lid and remove the first sheet in a firm sideways pulling motion.
- Always replace the blue cap to avoid evaporation of the alcohol.
 - Not intended to be used on medical devices and other therapeutic goods.
 - Use only on pre-cleaned surfaces.
 - To kill Bacteria and Fungi, wipe the pre-cleaned surface and let it dry for 10 minutes.

8. Additional information is listed in the Safety Data Sheet.

DO NOT USE ON SKIN.
DO NOT MIX WITH DETERGENTS OR OTHER CHEMICALS.
USE UNDILUTED.



For advice in an emergency, contact a Poisons Information Centre 131 126 (Aust) or 0800 764 766 (NZ).

WARNING:

Flammable solid. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from open flames – No smoking. Wash hands thoroughly after handling. Wear eye protection. Avoid breathing vapours. Use only outdoors or in a well-ventilated area.

FIRST AID:

If in eyes, hold eyelids apart and flush the eyes continuously with running water. Remove contact lenses. Continue flushing for several minutes until all contaminants are washed out completely. Continue flushing for several minutes until all contaminants are washed out completely. If eye irritation persists, get medical advice.

Sponsored in Australia by O&M Halyard Australia Pty Ltd; Tenancy 1, Level 1, Trinita 3, 39 Delhi Road, North Rude NSW 2113
For Enquiries or to order please call 1800 664 227 (For Australia Only)
For New Zealand, please call 0800 105 501.

Made in China



O&M Halyard Australia Pty. Limited